



## Presentation

The drying capsule is indicated for dehumidifying hearing aids and earmolds, in order to avoid breakdown due to moisture.

The drying capsule is a translucent plastic capsule of 30mm in diameter and 12mm in height, which contains crystals of orange silica gel, and closed by a cardboard seal.

The drying capsule is packed by 4, 6 or 8 each in an individual sealed bag.

Desiccant kits are also available. They are supplied with a specific cup.

## Properties

The drying capsule is a hygiene product specially developed to effectively absorb moisture.

The drying capsule meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

## Using advices

Place the capsule, face upwards, under the foam.  
Put the hearing aid on the foam with the battery compartment open.

Close the cup and let the capsule act all night long.

Remember to close the cup after each use.  
Change the capsule when it becomes almost white.  
The duration of use is about 1 month.

Daily use is recommended, preferably at night.  
Regular dehumidification increases the life of hearing systems and preserves their effectiveness.

## Precautionary statements

External use.  
Do not pierce the capsule.  
Do not swallow.  
Keep out of reach of children.

## Product composition

Silica gel