



Presentation

The cleaning wipe is a cleaning product for the maintenance of hearing aids and earmolds.

It removes earwax responsible for the decrease in sound quality, increase the life of hearing systems and preserves their effectiveness.

The cleaning wipe is presented in an individual pack (of 60 x 80 mm or 40 x 65 mm).

The cleaning wipes can be packed in cases of variable quantity (x10, 12, 15, 25, 30, 40, 50 or 60).

Properties

The specific formulation of the solution complies with the requirements of the bactericidal standard EN 1040 and the yeasticidal standard EN 1275.

The cleaning wipe meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

Using advices

Open the bag and remove the wipe.
Clean hearing aids or earmolds with a wipe.
Wipe the hearing aid with a soft, dry cloth.

It is recommended to clean hearing aids and earmolds every day.
The cleaning wipe is for single use.
Discard the cleaning wipe after use.

Precautionary statements

Warning: Flammable liquid and vapor. Keep out of reach of children. Keep away from heat, hot surface, sparks, open flames and other ignition sources. No smoking.

Product composition

- Alcohol
- Water
- Chlorhexidine digluconate
- Mint perfume
- Benzalkonium chloride
- ...

