



Presentation

The drying spray is a care product which facilitates drying of sound tubes and earmolds.

It is packaged in 1.69, 5.07 or 21.97 fl oz (50, 150 or 650mL) bottle, provided with a push button.

Properties

The drying spray has been developed for a quick and efficient maintenance of the sound tubes.

The product is totally dry and neutral. It does not damage components or acrylic resins.

It meets the requirements of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety requirements for health.

Using advices

Use short burst pressure to dry the earmold after cleaning. Blow through the hearing tube to remove any water remaining in the tube and earmold.

Precautionary statement

Warning:

Extremely flammable spray.
Contains gas under pressure, may explode if heated.
Keep out of reach of children.

Product composition

- Propel 45