



## Presentation

The cleaning cup is an essential accessory to be used with cleaning tablets.

It provides an adequate environment to ensure efficient cleaning of the earmolds through the action of the cleaning tablets.

It could be printed or labeled.

## Properties

- **Height:** 70mm
- **Diameter:** 55mm
- **Cup color:** Transparent (with a label on it)
- **Seal color:** White

The cup has a sieve to facilitate the removal of the earmolds at the end of the cleaning operation.

Lightweight, it is easy to transport.

The cleaning cup meets the requirement of Class I Medical Devices, according to the Regulation 2017/745/EU related to Medical Devices, while respecting environmental and safety for health.

As manufacturer of Medical Devices, MG Development makes sure of the origin of the raw materials of all its products. And so, all the materials and the coloring agents are complying with the directives and the existing standards (**REACH** for the restriction of chemical substances).

## Using advices

- Place the cleaning tablet into the cup then fill two thirds of the cup with warm water.
- Separation the earmold from the hearing aid and drop it into the plastic cup sieve.

**Never place the hearing instrument in the liquid.**

- Leave about 30 minutes.
- Then remove the earmold, rinse it with warm water.
- Dry completely before putting on the hearing aid.

A cleaning tablet is single use.

Discard the solution after each use.

It is recommended to perform a thorough cleaning of the earmold, once a week.

## Precautionary statements

External Use.

Do not swallow.

Keep out of reach of children.

## Product composition

- Polypropylène