



Presentation

The cleaning tablet is a cleaning product used to clean the earmolds.

The cleaning tablet removes earwax responsible for the decrease in sound quality, increase the life of the hearing system and preserves their effectiveness.

The cleaning tablet are presented in packs of 4 cleaning tablets in a 90 x 90 mm scored blister pack.

The cleaning tablets are packed by 3 or 5 blister packs (12 or 20 tablets).

Cleaning kits consisting of a specific cup, tablets and an air ball are also available.

Properties

The effervescent cleaning tablet composition is a latest technology specially designed to efficiently dissolve the earwax without damaging your earmold.

The cleaning tablet meets the requirements of Class I Medical Devices to the Regulation 2017/745/EU related to Medical Device, while respecting environmental and safety requirements for health.

Using advices

- Place the cleaning tablet into the cup then fill two thirds of the cup with warm water.

- Separation the earmold from the hearing aid and drop it into the plastic cup sieve.

Never place the hearing instrument in the liquid.

- Leave about 30 minutes.
- Then remove the earmold, rinse it with warm water.
- Dry completely before putting on the hearing aid.

A cleaning tablet is single use.

Discard the solution after each use.

It is recommended to perform a thorough cleaning of the earmold, once a week.

Precautionary statements

External Use.
Single use.
Do not swallow.
Avoid eye contact.
Keep out of reach of children.

Product composition

- Potassium Monopersulfate
- Sodium Bicarbonate
- Citric Acid
- Sodium Carbonate
- Sorbitol
- ...